




ClearNose™ Nasal Aspirator

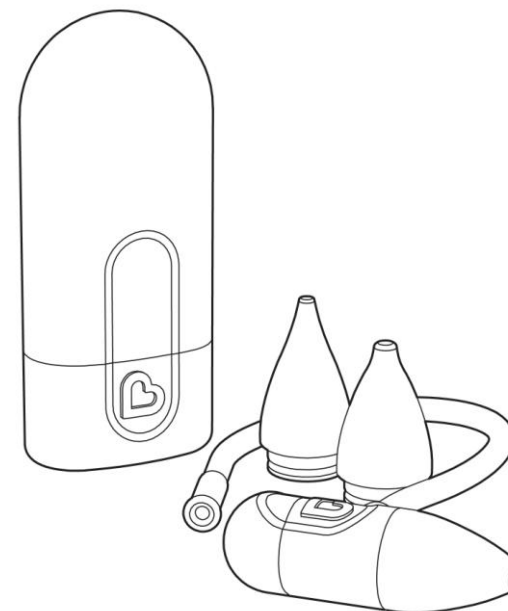

Reusable and hygienic—no filters required
Three different soft-silicone nose tips
BPA and phthalate free
Transparent reservoir to view expelled mucus
Convenient storage case


KEEP FOR FUTURE REFERENCE

!\ WARNING

- Refer to cleaning instructions before use.
- Use only for nasal aspiration. Never use for ears, mouth or eyes.
- Do not blow into mouthpiece while nose tip is placed in child's nose.
- Not a toy. Includes small parts. Keep away from children.
- Test suction before using aspirator on your child. Do not use if damaged.
- Do not use for nosebleed or on injured nose. Never insert nose tip too far into nose.
- Consult your physician on when and how often you should aspirate your baby's nasal passages.



NOTE: A small piece of tissue can be placed inside the mucus reservoir as a barrier to ease cleaning or for extra runny noses.

1. If necessary, spray or apply a few drops of nasal saline into each nostril to help loosen mucus.
 2. Gently press the nose tip into your child's nostril to create a seal.
- IMPORTANT:** Keep tube attachment upright to prevent mucus from contaminating air tube.
3. Inhale through the mouthpiece to suck mucus from your child's nose.
 4. Repeat for the other nostril.

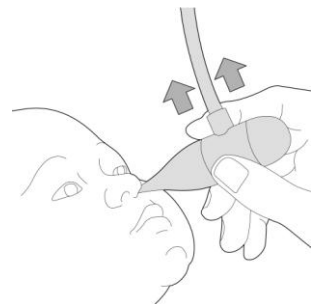

INCLUDED PARTS

Make sure all aspirator parts are joined correctly before use.

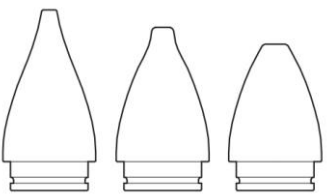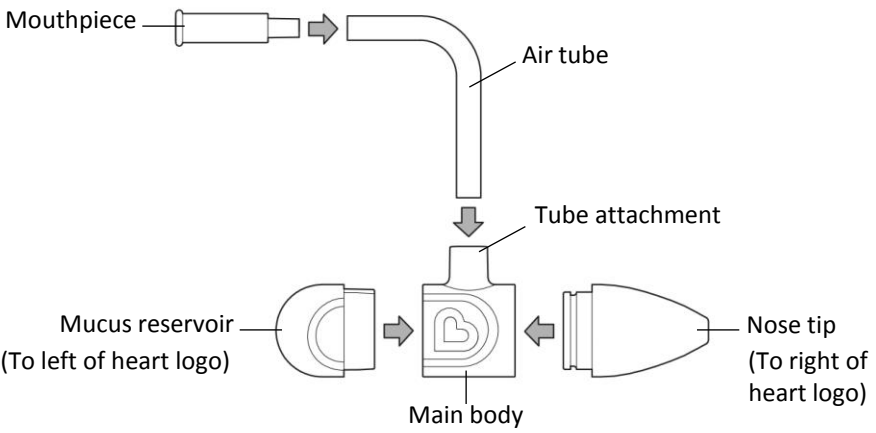

NOTE: Three different nose tips are included. Choose a tip that fits comfortably against your child's nostril to form a seal.


TO CLEAN

Take the aspirator apart for cleaning after each use. Wash with soap and water and rinse thoroughly. Do not microwave or steam sterilize.

Use a cotton swab to clean the nose tip. The air tube and mouthpiece can be cleaned with a few drops of rubbing alcohol.

After parts have been cleaned and dried, place the aspirator back into the storage case.

- 1. Disconnect the air tube to prevent kinks.

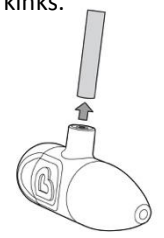


- 2. Wrap the air tube and the mouthpiece around the aspirator.




- 3. Slide the wrapped aspirator into the container.



- 4. Store the two extra nose tips in the container lid.

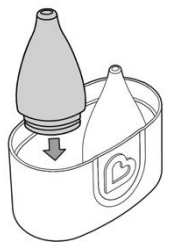

- 5. Place the lid onto the container to close.

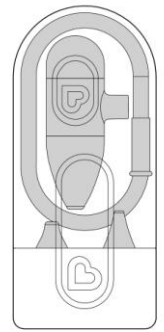

PRODUCT AND DESIGN ARE TM & ©2016 MUNCHKIN, INC. MUNCHKIN, THE HEART LOGO AND IT'S THE LITTLE THINGS ARE TRADEMARKS OR REGISTERED TRADEMARKS OF MUNCHKIN, INC. 7835 GLORIA AVE., VAN NUYS, CA 91406. MUNCHKIN BABY CANADA LTD., 50 PRECIDIO CT., UNIT A, BRAMPTON, ON L6S 6E3.

[CONTACT US icon - munchkin.com 1-800-344-2229]